



## DATA SHEET

Revision No.: ZJ0005  
Issue Date: Jun 1st, 2010

### Salmonella Real Time PCR Kit

Cat. No.: DD-0034-01



For use with LightCycler1.0/LightCycler2.0 real time PCR systems

*For in vitro Diagnostic use only*

### User Manual

European Authorized Representative (E.A.R.)  
Obelis S.A.  
34 Av. De Tervuren, bte 44  
B-1040 Brussels, Belgium  
Phone : +32.2.732.59.54  
Fax : +32.2.732.60.03  
E-mail : mail@obelis.net

#### 1. Intended Use

Salmonella real time PCR kit is used for the detection of Salmonella in stool or water samples by using real time PCR systems.

#### 2. Principle of Real-Time PCR

The principle of the real-time detection is based on the fluorogenic 5' nuclease assay. During the PCR reaction, the DNA polymerase cleaves the probe at the 5' end and separates the reporter dye from the quencher dye only when the probe hybridizes to the target DNA. This cleavage results in the fluorescent signal generated by the cleaved reporter dye, which is monitored real-time by the PCR detection system. The PCR cycle at which an increase in the fluorescence signal is detected initially (Ct) is proportional to the amount of the specific PCR product. Monitoring the fluorescence intensities during Real Time allows the detection of the accumulating product without having to re-open the reaction tube after the amplification.

#### 3. Product Description

Salmonella is a genus of bacteria that are a major cause of foodborne illness throughout the world. The bacteria are generally transmitted to humans through consumption of contaminated food of animal origin, mainly meat, poultry, eggs and milk. The symptoms of Salmonella infection usually appear 12–72 hours after infection, and include

fever, abdominal pain, diarrhoea, nausea and sometimes vomiting. The illness usually lasts 4–7 days, and most people recover without treatment. However, in the very young and the elderly, and in cases when the bacteria enter the bloodstream, antibiotherapy may be needed.

Salmonella real time PCR Kit contains a specific ready-to-use system for the detection of the Salmonella by polymerase chain reaction (PCR) in the real-time PCR system. The master contains reagents and enzymes for the specific amplification of the Salmonella DNA including *S. enterica*, *S. bongoru*, *S. subterranea* and suspecies of *S. enterica*. Fluorescence is emitted and measured by the real time systems' optical unit during the PCR. The detection of amplified Salmonella DNA fragment is performed in fluorimeter **channel FAM** with the fluorescent quencher BHQ1. DNA extraction buffer is available in the kit and excreta or water samples are used for DNA extraction.

In addition, the kit contains a system to identify possible PCR inhibition by measuring the HEX/VIC/JOE fluorescence of the internal control (IC). An external positive control ( $1 \times 10^8$  copies/ml) allows the determination of the gene load.

#### 4. Kit Contents

Ref.	Type of Reagent	Presentation 25rxns
1	DNA Extraction Buffer	2 vials, 1.5ml
2	Salmonella Reaction Mix	1 vial, 450µl
3	PCR Enzyme Mix	1 vial, 12µl
4	Molecular Grade Water	1 vial, 400µl
5	Internal Control (IC)	1 vial, 30µl
6	Salmonella Positive Control ( $1 \times 10^8$ copies/ml)	1 vial, 30µl

**Analysis sensitivity:  $1 \times 10^4$  copies/ml**

#### 5. Storage

- All reagents should be stored at -20°C. Storage at +4°C is not recommended.
- All reagents can be used until the expiration date indicated on the kit label.
- Repeated thawing and freezing (>3x) should be avoided, as this may reduce the sensitivity of the assay.
- Cool all reagents during the working steps.
- Reaction mix should be stored in the dark.

#### 6. Additionally Required Materials and Devices

- Biological cabinet
- Real time PCR system
- Desktop microcentrifuge for "ependorf" type tubes (RCF max. 16,000 x g)
- Vortex mixer
- Real time PCR reaction tubes/plates
- Cryo-container
- Pipets (0.5µl – 1000µl)
- Sterile filter tips for micro pipets
- Sterile microtubes

Gentaur Molecular Products  
Voortstraat 49  
1910 Kampenhout, Belgium

- Disposable gloves, powderless
- Biohazard waste container
- Refrigerator and Freezer
- Tube racks

## 7. Warnings and Precaution

- Carefully read this instruction before starting the procedure.
- For in vitro diagnostic use only.
- This assay needs to be carried out by skilled personnel.
- Clinical samples should be regarded as potentially infectious materials and should be prepared in a laminar flow hood.
- This assay needs to be run according to Good Laboratory Practice.
- Do not use the kit after its expiration date.
- Avoid repeated thawing and freezing of the reagents, this may reduce the sensitivity of the test.
- Once the reagents have been thawed, vortex and centrifuge briefly the tubes before use.
- Quickly prepare the reaction mix on ice or in the cooling block.
- Set up two separate working areas: 1) Isolation of the RNA/ DNA and 2) Amplification/ detection of amplification products.
- Pipets, vials and other working materials should not circulate among working units.
- Use always sterile pipette tips with filters.
- Wear separate coats and gloves in each area.

## 8. Sample Collection, Storage and transportation

- Collect samples in sterile tubes;
- Specimens can be extracted immediately or frozen at -20°C to -80°C.
- Transportation of clinical specimens must comply with local regulations for the transport of etiologic agents

## 9. Procedure

### 9.1 DNA-Extraction

DNA extraction buffer is supplied in the kit. Please thaw the buffer thoroughly and spin down briefly in the centrifuge before use.

#### 9.1.1 Stool samples

- 1) Take about 50mg samples to a 1.5ml tube; add 1.0ml normal saline then vortex vigorously. Centrifuge the tube at 13000rpm for 2 minutes, carefully remove and discard supernatant from the tube without disturbing the pellet.
- 2) Add 100µl DNA extraction buffer, close the tube then resuspend the pellet with vortex vigorously. Spin down briefly in a table centrifuge.
- 3) Incubate the tube for 10 minutes at 100°C.
- 4) Centrifuge the tube at 13000rpm for 5 minutes. The supernatant contains the DNA extracted and can be used for PCR template.

#### 9.1.2 Water samples

- 1) Take 3ml water to a tube, Centrifuge the tube at 13000rpm for 2 minutes, carefully remove

and discard supernatant from the tube without disturbing the pellet.

- 2) Add 100µl DNA extraction buffer, close the tube then resuspend the pellet with vortex vigorously. Spin down briefly in a table centrifuge.
- 3) Incubate the tube for 10 minutes at 100°C.
- 4) Centrifuge the tube at 13000rpm for 5 minutes. The supernatant contains the DNA extracted and can be used for PCR template.

### Attention:

**A.** During the incubation, make sure the tube is not open, as the vapor will volatilize into the air and may cause contamination if the sample is positive.

**B.** The extraction sample should be used in 3 hours or store at -20°C for one month.

**C.** Different DNA extraction kits are available. You may use your own extraction systems or the commercial kit based on the yield. For the DNA extraction, please comply with the manufacturer's instructions.

### 9.2 Internal Control and Positive Control

It is necessary to add internal control (IC) in the reaction mix. Internal Control (IC) allows the user to determine and control the possibility of PCR inhibition.


Add the internal control (IC) 1µl/rxn and the result will be shown in the HEX/VIC/JOE.

**Attention:** It is necessary to dilute the internal control and positive control supplied in the kit by 10 times with molecular grade water before detection, and close the tube immediately then vortex for 10 seconds.

Because of transportation with carbon dioxide ice, there may be white precipitate in tubes of internal control and positive control, but it will disappear in a few minutes when it is incubated at room temperature. Besides, the white precipitate have no effect on the detection result.

### 9.3 PCR Protocol

The Master Mix volume for each reaction should be pipetted as follows:

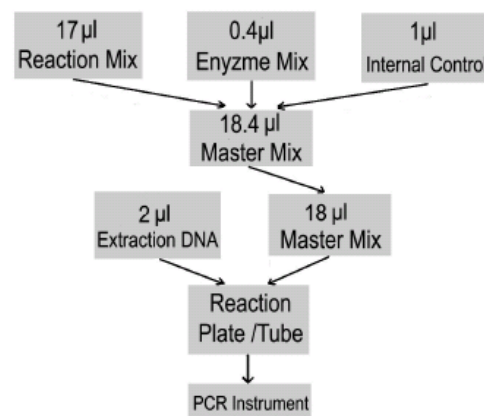

PCR system without HEX/VIC/JOE channel may be treated with 1µl Molecular Grade Water instead of 1µl IC.

1) The volumes of Reaction Mix and Enzyme Mix per reaction multiply with the number of samples, which includes the number of the controls, standards and sample prepared. Molecular Grade Water is used as the negative control. For reasons of unprecise pipetting, always add an extra virtual sample. Mix the master mix completely then spin down briefly in a centrifuge.

2) Pipet **18µl** Master Mix with micropipets of sterile filter tips to each *Real time* PCR reaction plate/tube. Then separately add **2µl** DNA sample, positive and negative controls to different reaction plate/tubes. Immediately close the plate/tubes to avoid contamination.

3) Spin down briefly in order to collect the Master Mix in the bottom of the reaction tubes.

4) Perform the following protocol in the instrument: **37°C for 2 min, 1 cycle; 94°C for 2 min, 1 cycle; 93°C for 5 sec, 60°C for 30 sec, 40 cycles.**

**Fluorescence is measured at 60°C; FAM and HEX/VIC/JOE channels should be chosen.**

**10. Baseline setting:** just above the maximum level of molecular grade water.

**11. Quality control:** The crossing point value of molecular grade water and positive control in FAM channel shows blank and  $\leq 35$  respectively; The crossing point value of internal control in HEX/VIC/JOE channel shows 25~33; otherwise the result is invalid.

## **12. Data Analysis and Interpretation**

The following results are possible:

1) The crossing point value in channel FAM shows  $\leq 35$ .

**The result is positive:**

**The sample contains salmonella DNA.**

2) The crossing point value in channel FAM shows 35~40, please repeat again. **If the result still shows 35~40, it can be considered negative.**

3) In channel FAM no signal is detected, at the same time, a HEX/VIC/JOE signal from the Internal Control appears.

**The sample does not contain any salmonella DNA. It can be considered negative.**

4) Neither in channel FAM nor in channel HEX/VIC/JOE signal is detected. **A diagnostic statement can not be made.** Inhibition of the PCR reaction.